



[www.SummitHandpieceExpress.com](http://www.SummitHandpieceExpress.com)

## Maintenance & Sterilization Guidelines

### HIGHSPEED HANDPIECES

1. After every use, unscrew handpiece from tubing
2. Over the sink, spray a shot of lubricant into the intake tube at the base of the handpiece. It's the smaller of the 2 tubes
3. Wipe off excess lubricant from handpiece
4. Install a bur and run for 30 seconds or until all excess oil exits the head
5. Insert into Autoclave bag and load sterilization trays – paper side up
6. Allow handpieces to dry in bags – paper side up

*\*ALWAYS lubricate handpieces prior to autoclaving.  
Do NOT lubricate the handpiece after sterilization.*



“Lube Free” Highspeed Handpieces: Lube Free Highspeed handpieces do not require lubrication. However, because no lubrication is being used, the turbine and housing O-rings eventually dry out and shrink causing turbine failures. The housing will eventually leak air and water. When this occurs, send the handpiece in for service.


**Questions? Call 877.342.0101**

### LOWSPEED HANDPIECES

**Motors** – A few drops of oil before every autoclave.  
Place lubricant in smaller of the 2 large holes



**Latch Angles** – A few drops of oil before every autoclave

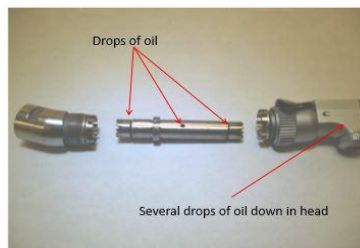

**Straight Nose Cones** – use high speed spray into nose cone to lubricate and prevent corrosion build up



**Servicing ALL makes and models of Highspeed, Lowspeed, Electric & Surgical handpieces!**